## ClinCal<sup>®</sup> - Calibrator

**Plasma-Calibrator, lyophilised, for Coenzyme Q10**

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The mean value was assigned with the ClinRep<sup>®</sup> complete kit for Coenzyme Q10 in Whole Blood, Plasma and Serum. The uncertainty of the mean value is  $\pm 1.6\%$  at a confidence level of 95%. The mean value is traceable to NIST-SRM968d. /

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### Intended use:

ClinCal<sup>®</sup> Plasma-Calibrators are used for calibration of the HPLC-system. This lyophilised calibrator has to be prepared like a patient sample.

### Reconstitution:

Add exactly 2.0 ml HPLC-water to the vial and mix for 15 min. When all material is dissolved, the solution is ready to use.

### Storage and stability:

Originally closed and stored below  $-18\text{ °C}$  in the dark, the lyophilised plasma-calibrator is stable for 12 months, but not longer than the expiration date printed on the label. After reconstitution the stability of the analyte is:

- at least 1 day when stored at  $15 - 30\text{ °C}$  in the dark
- at least 7 days when stored at  $2 - 8\text{ °C}$  in the dark
- at least 3 months when stored below  $-18\text{ °C}$  in the dark  
(avoid repeatedly freezing and thawing)

The average residual moisture of this lot is 1.03 %.

### Pack size:

ClinCal<sup>®</sup> Plasma-Calibrator for Coenzyme Q10: 5 x 2 ml, **order no.: 31013**

### Caution:

The human plasma that was used for manufacturing the calibrator was tested for the following infectious markers and found negative: HIV 1/2- and HCV-antibodies, hepatitis B-surface antigen, HIV- and HCV-genome (PCR). Nevertheless, the plasma-calibrator should be considered as potentially infectious and treated with appropriate care.

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