ClinCal® - Calibrator

Plasma-Calibrator, lyophilised, for Risperidone, Paliperidone

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The values were assigned with the ClinRep® complete kit for atypical Neuroleptics in Plasma/Serum.

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Intended use:

ClinCal® Plasma-Calibrators are used for calibration of the HPLC-system. This lyophilised calibrator has to be prepared like a patient sample.

Reconstitution:

Add exactly 5.0 ml HPLC-water to the vial and mix for 15 min. When all material is dissolved, the solution is ready to use.

Storage and stability:

Originally closed and stored at 2 - 8 °C, the lyophilised plasma-calibrator is stable for 36 months, but not longer than the expiration date printed on the label. After reconstitution the stability of the analyte is:

at least 3 days when stored at 15 - 30 °C

at least 7 days when stored at 2 - 8 °C

at least 3 months when stored below -18 °C

(avoid repeatedly freezing and thawing)

The average residual moisture of this lot is 1.25 %.

Pack size:

ClinCal® Plasma-Calibrator for Risperidone, Paliperidone: 5 x 5 ml, **order no.: 40113**

Caution:

The human plasma that was used for manufacturing the calibrator was tested for the following infectious markers and found negative: HIV 1/2- and HCV-antibodies, hepatitis B-surface antigen, HIV- and HCV-genome (PCR). Nevertheless, the plasma-calibrator should be considered as potentially infectious and treated with appropriate care.

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CHEMICALS + INSTRUMENTS GmbH Labortechnik