## ClinCal<sup>®</sup> - Calibrator

**Serum-Calibrator lyophilised, for Vitamin A and E**

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The values were assigned with the ClinRep<sup>®</sup> complete kit for Vitamin A and E. The uncertainty of the mean values is  $\pm 0.8\%$  for Vitamin A and  $\pm 1.6\%$  for Vitamin E at a confidence level of 95%.

The values are traceable to NIST-SRM968d.

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### Intended use:

ClinCal<sup>®</sup> Serum-Calibrators are used for calibration of the HPLC-system. This lyophilised calibrator has to be prepared like a patient sample.

### Reconstitution:

Add exactly 3.0 ml HPLC-water to the vial and mix for 15 min. When all material is dissolved, the solution is ready to use.

### Storage and stability:

Originally closed and stored at 2 - 8 °C in the dark, the lyophilised serum-calibrator is stable for 36 months, but not longer than the expiration date printed on the label. After reconstitution the stability of the analytes is when stored in the dark:

- at least 8 hours when stored at 15 - 30 °C
- at least 3 days when stored at 2 - 8 °C
- at least 3 months when stored below -18 °C  
(avoid repeatedly freezing and thawing)

The average residual moisture of this lot is 0.64 %.

### Pack size:

ClinCal<sup>®</sup> Serum-Calibrator for Vitamin A and E: 1 x 3 ml, **order no.: 22013**

### Caution:

The human serum that was used for manufacturing the calibrator was tested for the following infectious markers and found negative: HIV 1/2- and HCV-antibodies, hepatitis B-surface antigen, HIV- and HCV-genome (PCR). Nevertheless, the serum-calibrator should be considered as potentially infectious and treated with appropriate care.

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