ClinCal® - Calibrator

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Intended use:

ClinCal® Serum-Calibrators are used for calibration of the HPLC-system. This lyophilised calibrator has to be prepared like a patient sample.

Reconstitution:

Add exactly 0.5 ml HPLC-water to the vial and mix for max. 3 min. After that the lyophilisate is reconstituted and ready to use.

Caution: Immediately after reconstitution the serum-calibrator has to be prepared according to the instruction manual of the ClinRep® complete kit for vitamin C and analysed.

Storage and stability:

Originally closed and stored at -20 °C in the dark, the lyophilised serum-calibrator is stable for 12 months, but not longer than the expiration date printed on the label. After reconstitution, the serum-calibrator has to be used immediately and cannot be stored. The average residual moisture of this lot is 1.34 %.

Pack size:

ClinCal® Serum-Calibrator for Vitamin C: 5 x 0.5 ml, **order no.: 28013**

Caution:

The human serum that was used for manufacturing the calibrator was tested for the following infectious markers and found negative: HIV 1/2- and HCV-antibodies, hepatitis B-surface antigen. Nevertheless, the serum-calibrator should be considered as potentially infectious and treated with appropriate care.

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