ClinCal[®] - Calibrator

Serum Calibrator Set lyophilised / Serum-Kalibrator-Set lyophilisiert

FOR 25-OH-VITAMINS D2/D3 / FÜR 25-OH-VITAMINE D2/D3

Intended use:

ClinCal[®] serum calibrators are used for calibration of the analysis-system. These lyophilised calibrators are based on horse serum and are available as a calibrator set with 4 levels, including a blank. The calibrators have to be prepared like patient samples in one series of analyses.

Reconstitution:

Add exactly 1.0 ml of HPLC-water to the vial and mix for 15 min. When all material is dissolved, the solution is ready to use.

Storage and stability:

Originally closed and stored below -18 °C in the dark, the lyophilised serum calibrators are stable for 36 months, but not longer than the expiration date printed on the label.

After reconstitution the stability of the analytes is:

- at least 3 days when stored at 15 - 30 °C in the dark
- at least 7 days when stored at 2 - 8 °C in the dark
- at least 3 months when stored below -18 °C in the dark (avoid repeatedly freezing and thawing)

Notes:

The mean values of the analyte are chosen in a way that a linear calibration curve can be generated over a suitable range of concentration. According to quality assurance all ClinCal[®] calibrators have to pass strict quality control procedures during manufacturing. RECIPE guarantees the same stability and constitution for each vial of one lot. The variation of the filling volume is max. +/- 0.25 %. The average residual moisture of this lot is 0.86 %.

Mean values:

The mean values were established in additional independent reference laboratories with supervision of RECIPE, according to the guideline of the German Medical Council (RiliBäk) with statistic methods and are traceable to NIST-SRM972.

Pack size:

ClinCal[®] Serum Calibrator Set for 25-OH-Vitamins D2/D3 Level 0 - 3
4 x 1 x 1 ml, **order no.: MS7013**

Precautions:

This ClinCal[®] Calibrator was manufactured from pooled horse serum. It should be considered as potential infectious and treated with appropriate care.

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CHEMICALS + INSTRUMENTS GmbH Labortechnik